Lomatuell® H

Salbentüll, hydrophob

Tulle gras vaseliné, hydrophobe

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<p>Instructions for use</p>

Composition:

Knitted, wide-mesh, net-like tulle produced from 100 % cotton, impreg- nated with a hydrophobic (water-resistant) ointment basis (white paraffin BP).

Indications:

Lomatuell® H is used for the treatment of skin donor sites, leg ulcers, decubitus ulcers, burns, cuts, abrasions.

Contraindications:

Contraindications have not been observed.

Side effects:

Side effects have not been observed.

Interactions with other preparations:

Interactions have not been observed.

Dosage and instructions for use:

Prior to application, the wound should be cleaned as usual. Lomatuell® H should then be applied overlapping the wound edges by approximately 1 cm. With heavily exuding wounds, a high absorbency dressing should be applied on top (e.g. Vliwazell®), with moderately to slightly exuding wounds, a gauze or nonwoven dressing (e.g. Gazin® or Vliwasoft®) will be sufficient. To assist a moist warm wound environment and to protect the wound against germs and bacteria from the outside, the dressing can be secured with a semi-per- meable film (e.g. Suprasorb® F). On infected wounds, only air permeable

dressng retention materials should be used (e.g. Curafix® H).

Dressing change:

After liquefaction and absorption of the Vaseline (Paraffin), Lomatuell® H should be removed. As a rule, this effect takes approximately 1-3 days.

Important information!

Lomatuell® H should not be stored above 25 °C. Lomatuell® H Paraffin Gauze Dressings are sterile, unless the package is opened or damaged. Do not resterilize.

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